

Common Collection Devices

Always indicate source for Microbiology specimens

Aptima® Multitest Swab

Order Number: 177680

Collection for:

- Gonorrhoeae by TMA-GCRNAB
- Gonorrhoeae/Chlamydia by TMA-GCCTMA
- Chlamydia by TMA-CTRNAB
- BVTMA (Bacterial Vaginosis)
- VAGCOM – Vaginitis Comprehensive Panel
- CVTTMA (Candida vaginitis & Trichomonas)



Aptima® Unisex Swab

Order Number: 177671

Collection for:

- Gonorrhoeae by TMA-GCRNAB
- Gonorrhoeae/Chlamydia by TMA-GCCTMA
- Chlamydia by TMA-CTRNAB



Pinworm

(Paddle Collection)

Order Number: 177784

Collection for:

- Pinworm Exam - PINWRM



Myco F/Lytic Bottle

Order Number: 180274 (Only order when needed)

Collection for:

- Culture, Fungal (Blood) - CULFUB
- Culture, Acid Fast (Blood) - CUAABL



Green Top Tube (10mL)

Submit one (1) 10mL tube

Order Number: 179216

Collection for:

- T-Spot - TSPOT



Blood Culture Bottle

Order Number:

Aerobic BCB: 176901

Anaerobic BCB: 176900

Collection for:

- Blood Culture - CULBLD



BD Affirm™ Ambient Transport

Order Number: 176903

Collection for:

- BV/Vaginitis Panel - BVVAF



Culture Swab (Teal or White Cap)

(Only send one collection device, either is suitable) Order Number: 177091

Collection for:

- Cervical/Vaginal Culture - CULCX
- Group B Strep Screen - CULSTR
- Group B Strep Screen (PCR) - GBSPCB
- Group B Strep Screen - reflex to sensitivity - GBREB
- Miscellaneous Source Culture, Aerobic - CULMIS
- Throat Culture - CULTHR
- Urogenital Culture - CULGEN
- Wound Culture, Aerobic - CULWND
- Yeast Culture - CULYST
- Neisseria Gonorrhoeae Culture - CULGC



InSure™ Enterix FIT

Occult Blood-InSure FIT (Fecal immunochemical test)

Order Number: 178105

Collection for:

- Screening test - INSURS
- Diagnostic - INSURD



Pediatric Blood Culture Bottle

FOR HOSPITAL USE ONLY

Contact account rep for assistance

Collection for:

- Blood Culture - CULBLD
- Children under 12 ONLY



Common Collection Devices

Always indicate source for Microbiology specimens

Liquid-Based Pap Vial

Order Number:

Thin Prep® Vial: 178156 Brushes/Scrapers: 41629 Broom: 178058

Brushes: 178032 Scrapers: 177612

Collection for:

- PAP liquid based THINPREP - 73717
- PAP w/REFLEX HPV - 73783
- PAP w/REFLEX HPV, INCLUDING CHLAMYDIA/GC - P75282
- PAP w/REFLEX HPV w/REFLEX GENOTYPE, CHLAMYDIA/GC/TRICH - P76642
- PAP w/REFLEX HPV w/ REFLEX GENOTYPE, CHLAMYDIA/GC - P75344
- PAP w/REFLEX HPV w/REFLEX GENOTYPE - P75283
- PAP w/HPV, CHLAMYDIA/GC/TRICH - P75776
- PAP w/HPV, CHLAMYDIA/GC - P74510
- PAP w/HPV - P75271
- PAP w/HPV w/REFLEX GENOTYPE, CHLAMYDIA/GC/TRICH - P76645
- PAP w/HPV w/REFLEX GENOTYPE, CHLAMYDIA/GC - P75280
- PAP w/HPV w/REFLEX GENOTYPE - P75281
- PAP w/CHLAMYDIA/GC/TRICH - P75775
- PAP w/CHLAMYDIA/GC - P75279
- Molecular testing off Thin Prep Vial
- Chlamydia/N.gonorrhoeae - GCCTMA
- Chlamydia/N.gonorrhoeae/Trichomonas - GCT
- Chlamydia/N.gonorrhoeae/Trichomonas/HPV HR - GCTHRP
- HPV HR - HPVHR
- HPV HR reflex to genotyping - HPVHR



Ova/Parasite Kit (Para Pak® Pink and Blue)

Order Number: 40542

Collection for:

- Cryptosporidium - CRYSA
- Fecal Leukocyte - LEUKF
- Giardia Antigen - GIARAG
- Ova and Parasite routine - OVAPR
- Ova parasite: Traveler's Screen - OP1
- Traveler's Screen - OP2
- Traveler's Screen - OP3



Uri-Tube (Yellow or Red/Yellow Speckled)

Only send one collection device, either is suitable

Order Number:

Straw: 61940 Tube: 177834

Collection for:

- Urinalysis



Urine Culture Transport (BD Gray Cap)

Order Number:

Tube: 32543

Collection for:

- Urine Culture - CULURN



One urinalysis tube + One culture tube is needed for urinalysis + culture

Stool Culture Transport (Para Pak® Orange)

Order Number: 33316

Collection for:

- Stool Culture - CULSTL
- Yersinia Culture - CULYER
- Vibrio - CULVIB
- Gastrointestinal PCR Panel (FOR HOSPITAL USE ONLY)



Sterile Cup

Order Number: 178188

Collection for:

- Clostridioides Difficile Ag (Refrigerate) - CDIFF
- Cytomegalovirus Urine (Refrigerate) - CMVDNA
- Fungus Culture - CULFUO
- Helicobacter Stool Antigen - HPSAG
- Rotavirus Antigen - ROTAV
- Mycobacterium Culture - CULAFB
- Hemocult Screening - OCCSCR
- Hemocult Diagnostic - OCCDIA



Common Collection Devices

Always indicate source for Microbiology specimens

Viral Transport Medium (VTM)

Order Number: 176819

Collection for:

- COVID SARS CoV-2 RNA – COVID19
- Herpes Simplex Virus (PCR) – HSVPCR
- Varicella Zoster Virus DNA – VZVPCB
- Viral Culture/PCR – Refer to CompuNet Directory of Services
- Influenza A/B - FLUPCR
- Bordetella Pertussis/Parapertussis PCR – BORPCR
- RSV (PCR) – RSVPCB



Dual Swab

(Yellow or Red Cap)

Order Number: 177092

(Only send one collection device, either is suitable)

Collection for:

- MRSA - MRSAP
- Candida auris PCR - CAPCR



QuantiFERON

FOR HOSPITAL USE ONLY

Contact account rep for assistance

Collection for:

- QuantiFERON - QTF



Glucocrush™

Order Number:

Orange 50g: 57434

Lemon Lime 50g: 57432

Fruit Punch 50g: 63167

Orange 75g: 33296

Lemon Lime 100g: 51491

Fruit Punch 100g: 177831

Orange 100g: 57427

Collection for:

- Glucose Tolerance Testing



Formalin Cup

Order Number:

Large: 178138

Small: 178164

Collection for:

- Biopsy Tissue - See online DOS
- Histology



24-Hour Urine Collection

Order Number: 177811

Collection for:

- 24 hour urine collection



HPV Self-Collection Kit (Swab + Vial)

Order Number: Contact account rep for assistance

Collection for:

- HPV Self Collect w/ Genotype - HPVSC



One self-collect swab + one Pap vial is necessary for HPV self-collect with Genotype

Cytolyt® Fixative

Order Number: 53386

Collection for:

- Cytology-NON GYN - 74315



Anaerobic Transport Tube

Order Number: 177840

Collection for:

- Anaerobic Culture - CULANA



Common Collection Devices

Always indicate source for Microbiology specimens

For Information:

Ike Northern

Infectious Diseases & Immunology

937-297-8334 | winorthern@compunetlab.com

Ed Bonvillain

Cytology

937-208-5050 | embonvillain@compunetlab.com